

STEP 1

REMOVE the Tyvek lid from the blister pack and transfer the contents to the sterile field.



STEP 2

ATTACH desired tubing to the LipiVage® harvester and to the waste container of the aspirator.



STEP 3


REMOVE the Luer-to-Luer transfer adapter and retain it for transferring fat later. Slowly retract the plunger to extend it to act as a handle.



STEP 4

CONNECT THE CANNULA:

Kit Version with Disposable Cannula: Place the cannula on the harvester and turn until the wings on the cannula engage with the locking mechanism; a "clicking" sound may be heard.

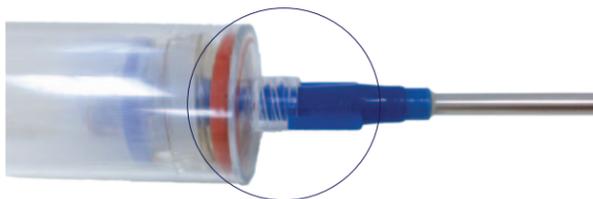


Without the Disposable Cannula: Attach the desired reusable cannula, ensuring a tight connection.

STEP 5



SET THE ASPIRATOR to 10-18 inHg (253-457 mmHg). With the plunger fully retracted, **HARVEST** for approximately one (1) minute.

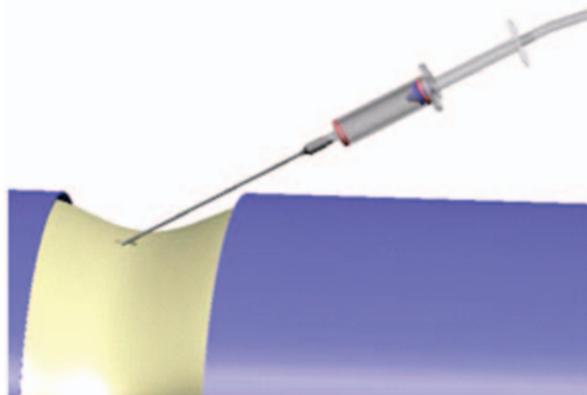

STEP 6




HOLD UPRIGHT and drain fluids.




STEP 7

Turn the aspirator off and **TRANSFER THE HARVESTED TISSUE** per method (A) or (B):

A Attach the supplied Luer-to-Luer transfer adapter to the distal end of the harvester and connect the receiving syringe to the other end of the adapter with the plunger fully inserted. Transfer the tissue from the harvester to the receiving syringe by pushing the plunger in while simultaneously pulling out the receiving syringe plunger (or, if easier, you may remove the plunger from the receiving syringe and replace when the transfer of tissue is complete).

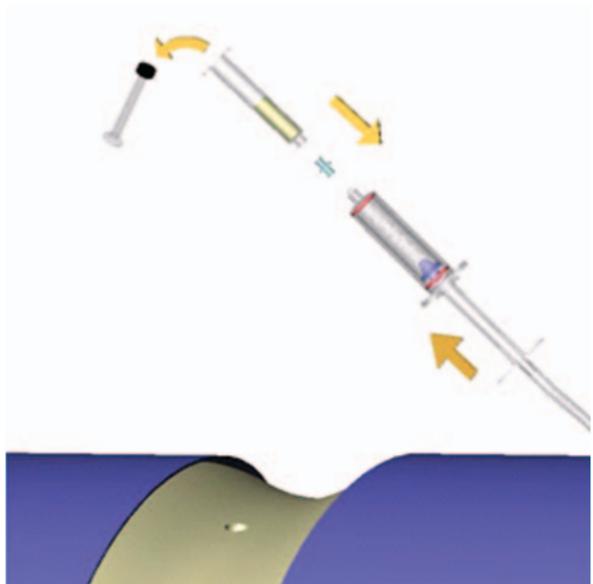

B Transfer the tissue from the harvester into the back of the receiving syringe with the plunger out. Once the transfer is complete, replace the plunger in the receiving syringe.

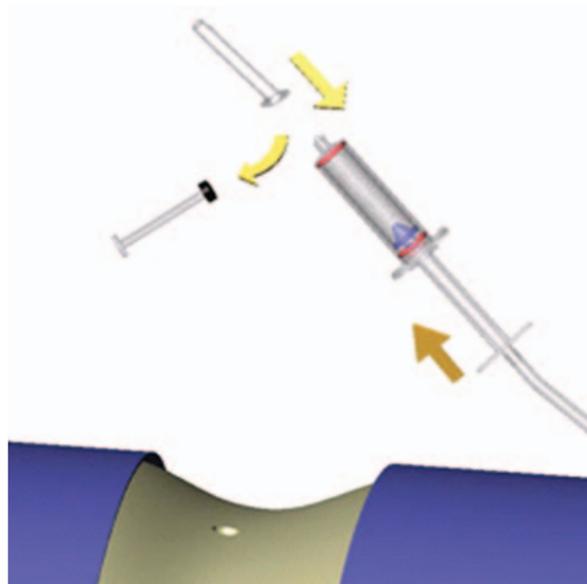

STEP 8

If additional tissue is desired, **CLEAN THE FILTER** prior to the next harvest by moving the plunger back and forth several times with the vacuum on. You may also flush the filter with sterile saline solution with the vacuum level on to remove oils from the filter walls.

Repeat steps 4 through 7.

