## ENGLISH

### Device Description:

LipiVarge® is a hollow barrel harvesting syringe with plunger and an in-line polyester filter that is intended to receive, concentrate and filter autologous adipose tissue from lipoplasty procedures, for re-implantation of the extracted tissue back into the same patient at other anatomical locations in their body using a piston syringe and injection cannula.

At one end of the LipiVarge® harvesting syringe there is a male connector (nozzle) for fitting the female connector (hub) of a cannula for the harvesting of fat cells, while at the other end of the device there is a universal tubing connection to attach to a tubing set (not supplied by Genesis Biosystems) that is connected to a controlled vacuum source, such as an aspirator or wall suction in the user facility.

The LipiVarge® harvesting syringe incorporates an in-line filter so there is no need to centrifuge, decant or expose fat cells to unnecessary handling. The device is for professional use only, made from medical grade polymers, single use, and provided sterile to the user.

### Intended Use(s):

LipiVarge® is used for aspiration, harvesting, filtering, and transferring of autologous adipose tissue for aesthetic body contouring. The system should be used with a legally marketed vacuum or aspirator apparatus as a source of suction. If harvested fat is to be re-implanted, the harvested fat is only to be used without any additional manipulation. LipiVarge® is intended for use in the following surgical specialties when aspiration of soft tissue is desired: plastic and reconstructive surgery, neurosurgery, gastrointestinal and affiliated organ surgery, urological surgery, general surgery, orthopedic surgery, gynecological surgery, thoracic surgery, and laparoscopic surgery.

### Intended for Use By:

Recommended for use by or under the supervision of a licensed physician for tissue irrigation purposes.

### Caution:

- US Federal law restricts this device to sale by or on the order of a physician.
- Inspect product packaging prior to opening. Do not use if damage is suspected. Only contents inside the tray are sterile. Outside surfaces of tray are NOT sterile.
- After use, dispose of device as a biohazard waste as per your local regulation requirements.

### To Set-Up Device:

- While holding the tray with one hand, peel back tray lid. Carefully remove device(s) from tray without touching outer surfaces.
- Remove the transfer adapter from tip of harvester and set aside.
- Attach tubing (ID size range from 0.25 to 0.375", which is not supplied) to plunger end and connect to appropriate vacuum or aspirator waste canister (not supplied) and set vacuum to recommended level of 20 in Hg or 508 mm Hg.
- If using provided disposable cannula, attach and tighten cannula until wings "click" past syringe bumps. If using other cannula, attach a luer lock style cannula and hand tighten until fully secure.

### To Harvest Fat:

- With plunger fully retracted, turn on vacuum suction and aspirate tissue from selected harvest site for approximately 60 seconds. It is recommended to not exceed 90 seconds.
- With plunger still fully retracted, hold device upright to drain fluids for approximately 10 seconds. Move plunger slightly to aid drainage if needed.
- Detach the cannula and turn off the vacuum.

### To Transfer Fat:

- Remove the plunger from the receiving syringe and slowly push the plunger of the harvesting device forward transferring tissue into the open back end of the receiving syringe. Replace receiving syringe plunger. Repeat with additional syringes.
- Attach the luer-to-luer transfer adapter supplied with the harvester to the distal end of the harvesting device and connect it to a receiving syringe with its plunger fully inserted.
- Slowly push the plunger on the harvesting device, pushing the harvested tissue into the receiving syringe while simultaneously pulling back the plunger of the receiving syringe, or remove the plunger of the receiving syringe and replace once full of harvested tissue.
- Repeat with additional syringes as required.

Should additional volume be required, remove the transfer adapter tip (if used) and re-connect the harvesting cannula. Rinse the filter walls by aspirating sterile water for several seconds. Stroke the plunger several times and then repeat Harvesting and Transfer steps above until the desired amount of tissue is harvested and transferred.

### Warnings:

- This device will not, in and of itself, produce significant weight reduction.
- This device should be used with extreme caution in patients with chronic medical conditions, such as diabetes, heart, lung, or circulatory system disease or obesity.
- The volume of blood loss and endogenous body fluid loss may adversely affect intra and/or postoperative hemodynamic stability and patient safety. The capability of providing adequate, timely replacement is essential for patient safety.

### Precautions:

- This device is designed to remove localized deposits of excess fat through small incisions and subsequently transfer the tissue back to the patient.
- Use of this device is limited to those physicians who, by means of formal professional training or sanctioned continuing medical education (including supervised operative experience), have attained proficiency in suction lipoplasty and tissue transfer.
- Results of this procedure will vary depending upon patient age, surgical site, and experience of the physician.
- Results of this procedure may or may not be permanent.
- The amount of fat removed should be limited to that necessary to achieve a desired cosmetic effect.

### Potential Adverse Events:

- Should there be an interruption in fat aspirating into the device:
- Turn off the vacuum suction and stroke the plunger several times to clear the filter walls.
  - Turn the vacuum suction on and continue the harvesting procedure.
  - If a clog occurs due to fibrous tissue entering the device during transfer:
    - Do not exert extreme force on the plunger as it may result in a rupture of the inner filter.
    - Instead, insert a hemostat through the device orifice to remove the clog.

- Should the cannula become loose using a durable cannula during the procedure:
- Turn off the vacuum suction and re-connect the cannula at the threaded interface using a hemostat to tighten the threads until firmly seated.

- Should the disposable cannula become loose during the harvesting procedure:
- Turn off the vacuum suction and re-connect the cannula at the threaded interface by hand tightening the threads until wings "click" past syringe bumps.

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